


Devolux[®] VITAL

Poly-L-lactic Acid & Hyaluronic Acid

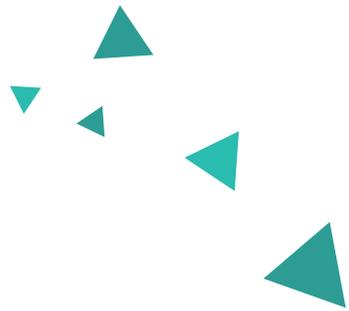

CONTENTS

01 Product Introduction

02 Instructions

03 Treatment Cases


04 FAQ






01

Product Introduction

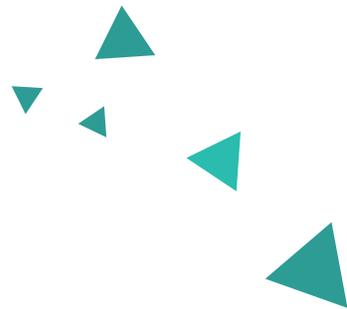

Introduction of Devolux® Vital

Devolux® Vital is a poly-L-lactic acid (PLLA) injectable in liquid form, as a collagen stimulator, Devolux® Vital helps gradually replace lost collagen, restore the deep, underlying structure of the skin to diminish facial wrinkles, for results that can last up to two years. Injectable hyaluronic acid applied in Devolux® Vital to reduce the appearance of wrinkles and facial folds, results are seen immediately.

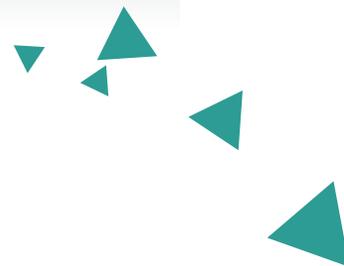


Function mechanism



1 Before injection


Depression before treatment.

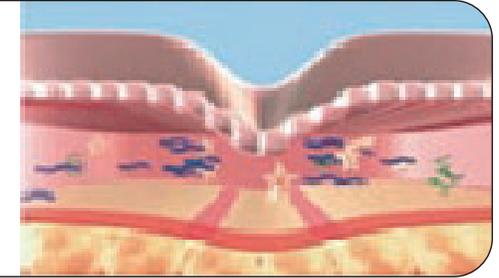

2

Immediately after injection

The optimal formula of HA+PLLA can fill in the depression immediately after injection.

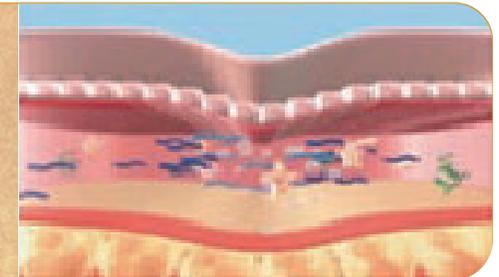

3

1-15 days after injection


Within 15 days of the treatment, the hyaluronic acid propped up the depressions, and the PLLA and the PLLA molecules were absorbed by the body.



4

30 days after injection


Devolux® Vital stimulates the underlying skin to rebuild collagen structure and collagen production.

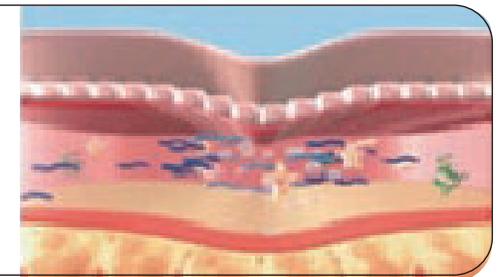



02

Instructions

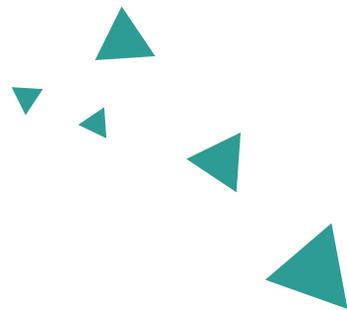

Mixing Method of Devolux® Vital

Name	Devolux Vital
Specification	5ml/ pc (160mg Poly-L-Lactic Acid+ 127mg hyaluronic acid)
Suggested mixing ratio	Add 0.9% saline 5ml
	Slightly shake the bottle to dissolve evenly
	Add lidocaine 2ml in 3-5 minutes
	it's recommended to use 25-27g sharp needle or 23g-25g blunt cannula for the injection
Injection layer	Between subdermal and superficial fat
Treatment session	2 times treatment is a session
	The second treatment is 1-1.5 months after the first treatment.
Duration	A session lasts for 2- 3 years
Expiry time	2 years
Storage environment	0-20 °C room temperature storage

Face

Neck

Vagina

Hand

Body

Buttock

Breast

Facial plumping
Firming and lifting
Contour adjustment
Improve skin tone
Improve acne marks

Sagging skin
Wrinkle


Vulva
Vagina
G point

Skin scars
Wrinkle



Sunken skin
Scar
Pregnancy striae

Enlargement

Enlargement

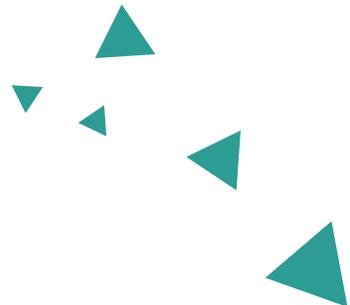


Application Areas	Volume	Injection Level	Remarks
Forehead 	2.5-10ml	Galeal layer - This layer is loose, has no nerves and blood vessels, and does not bruise.	If the amount is too large, the water will flow down.
Temple 	2.5-5ml	Subcutaneous fat layer - very loose layer, this is the avascular area, very good treatment.	Withdraw the needle and press until the medicine is evenly dispersed, then use a blunt needle to spread PLLA at the subcutaneous fat evenly, and press evenly.
Cheekbone 	2.5-5ml	Subcutaneous fat layer, on the SMAS fascia layer, do not break through the SMAS fascia layer.	
Tear through 	2.5-3ml	Close to the periosteum.	
Nasolabial fold 	2.5-3ml	Sharp needles are used in the pyramid style, and there are more drug delivery points near the periosteum layer.	Try not to make exaggerated expressions for three days after injection, to void muscle backlog and spread the medicine. upward In addition, it is recommended to use sharp needles vertically for the nasal base.

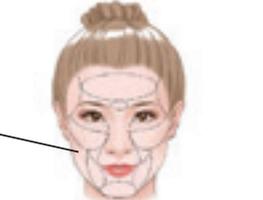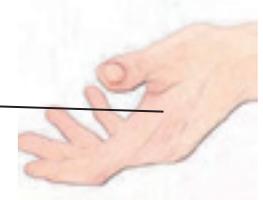Application Areas	Volume	Injection Level	Remarks
Cheek 	0.3-1ml on one side	Galeal layer - This layer is loose, has no nerves and blood vessels, and does not bruise.	It is recommended to use blunt needles flatly in multiple layers, subcutaneous, fat layer, fascia layer, subfascial tissue and periosteum.
Hand 	10-20ml on one side	Subcutaneous.	
Vagina	2-3ml on one side of vagina 1ml on one side of labia majora	On the mucous membrane of both sides of the vagina, by multi-point injection or one-point large volume injection.	Do not insert the needle deeply.
Neck 	1.5-3ml per wrinkle	30G sharp needle or 27G blunt needle, between subdermal and platysma muscle.	Mixing ratio: Add 15ml sterile water and 2ml Lidocaine.
Stretch marks 	20-30ml	At the obvious part of the fractured layer. The sharp needle is inserted vertically into the deep dermis.	Do not inject at the fat layer, Mixing ratio; Add 15ml sterile water and 2ml Lidocaine.


Suggestions

- 1 The nurse with the station should ensure that the medicine is evenly dissolved. Moderately oscillate for 5 minutes before treatment, observe the state of the drug in the needle tube, it must be evenly distributed, and cannot be injected in a precipitated state;
- 2 Do not wipe the needle with gauze during the treatment.
- 3 Large-scale disinfection of the treatment area.
- 4 Acupuncture withdrawal.
- 5 Be sure to stop the administration before the needle is with drawn, and focus on massaging at the needle entry point to spread the medicine, so as to avoid excessive drug residue at the needle entry point and increase the risk of nodules

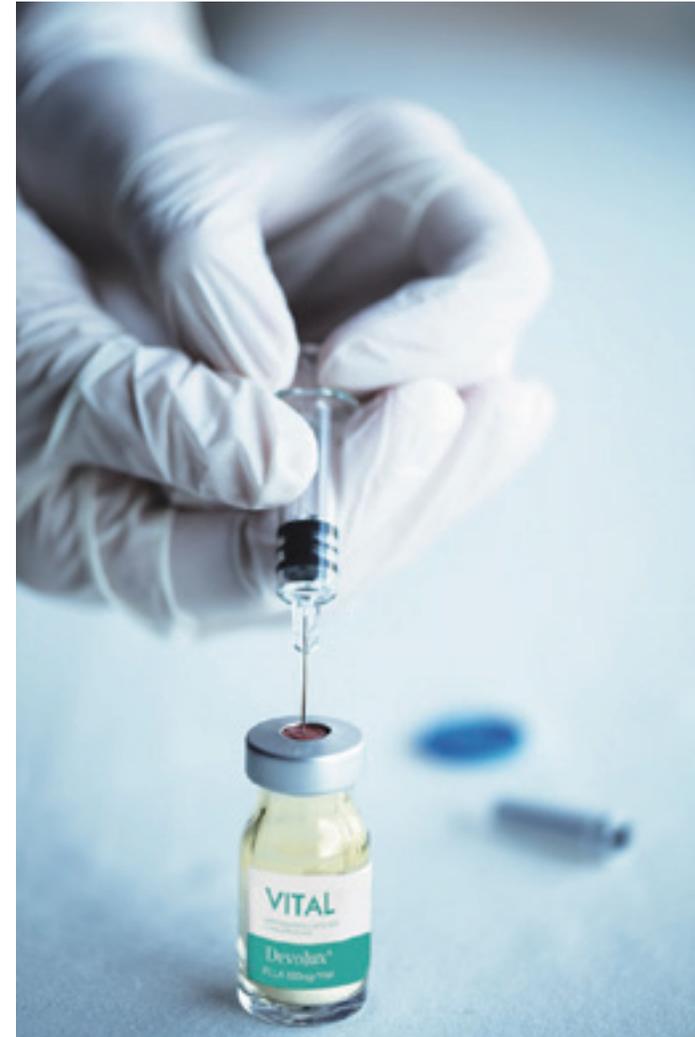



- 6 Pay attention not to excessive treatment, just reach the ideal state, do not overdose, reduce the occurrence of swelling;
- 7 If large-scale peeling is carried out, it may cause short-term skin dullness and swelling, which has nothing to do with the product itself. Tongyan needle has the auxiliary effect of brightening skin tone and improving skin quality
- 8 If the treatment of static lines wants to achieve better results, it is necessary to assist the positioning of Botox to ensure that the drug does not move and exert its curative effect in the right place
- 9 The effect of Devolux Vital is a gradual process. The onset time and duration of the effect are related to the customer's own condition, sensitivity to the drug, as well as the concentration, dose, and level of injection, and comprehensive evaluation is required.

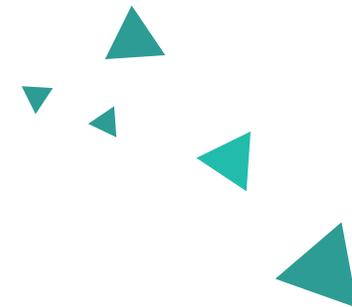

Postoperative nursing


- After treatment, apply ice several times within 24 hours to help reduce swelling.
- On the 4th day after the injection, massage the treatment area by yourself for 3 consecutive days. 3 times a day. 3 minutes each time.
- Avoid exposure to sunlight and UV lights



Contraindications




- ✓ Minors under the age of 18.
- ✓ People with local inflammation or infected skin.
- ✓ Those who are allergic to pharmaceutical ingredients.
- ✓ Those who have used anticoagulant drugs before surgery.
- ✓ Women during pregnancy, lactation and menstruation.
- ✓ People with scar constitution should be used with caution.

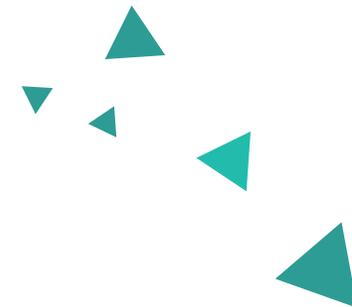





03


Treatment cases

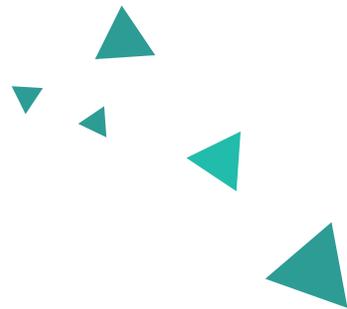


Cheek & Nasolabial fold

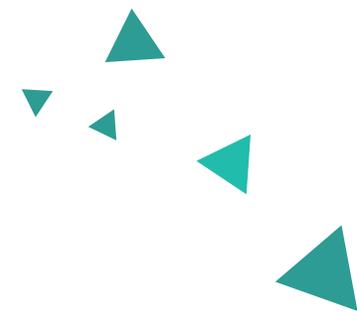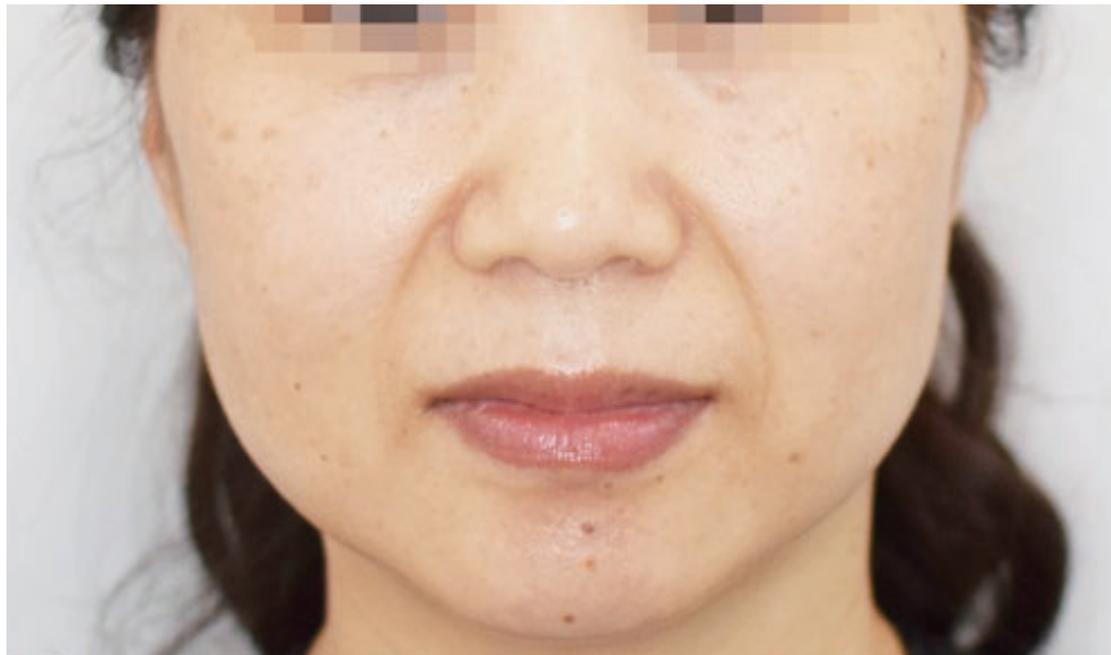



Nasal base & Cheek

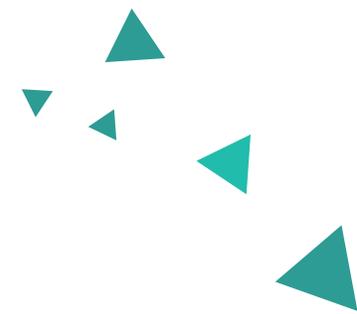


Cheek & Nasolabial fold

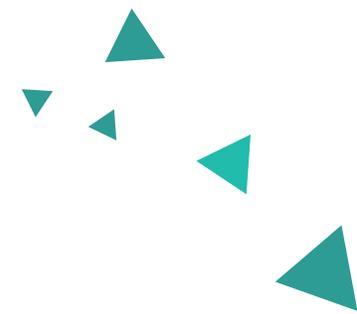


Nasolabial fold

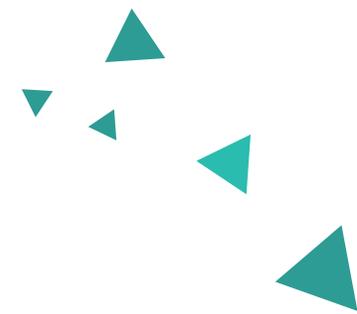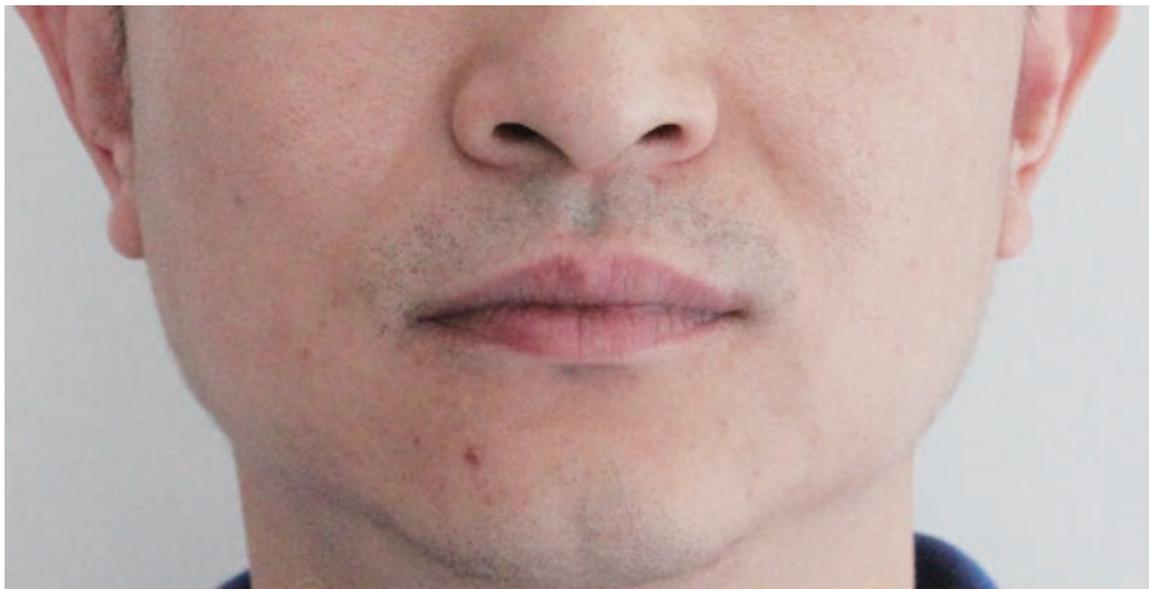

Temple

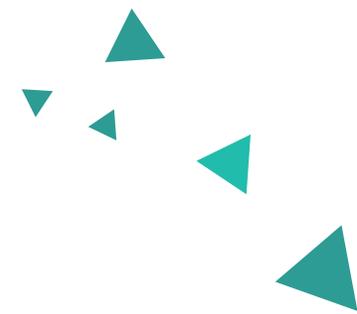

04 FAQ

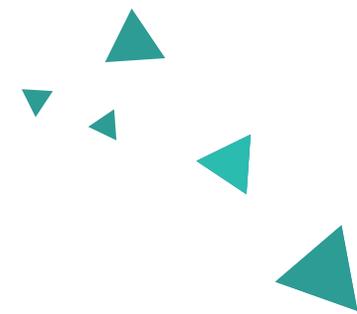

Devolux® VITAL

MAKE NATURE LONG LASTING



FAQ

- *How to inject to get a better effect?*

The blunt needle is stripped, and the suspension is injected widely with multiple layers and multiple tunnels. The blunt needle peeling method is easier than the sharp needle treatment, and the treatment effect will be better; the tissue swelling caused by the peeling will generally be relieved after 72 hours, and the patients can also be recommended to take the "Caomuxiliu Infusion Tablet" produced in Japan, also known as "Eliminate". After injection, massage (use the palm root position) to make the product relatively fixed in the interstitial space. After massage, 20-30 units of Botox injection can be used to assist positioning



- ***Can I exercise after surgery?***

Exercise can be resumed the next day, and the product has good stability. The recovery period of the human body after treatment is very short, and it will not touch water within 6-24 hours after treatment. The risk of infection mainly comes from the skin channel (pinhole) during treatment.

- ***Why it can't be mixed well?***

Long time exposure to high temperature in transportation or storage environment, or abnormal shaking such as violent shaking may cause product agglomeration. You can add normal saline and let it stand for 10 minutes to see the dilution effect; if there is still a lot of sediment, you can take the supernatant for treatment; such as a doctor If it is judged that it cannot be used, it can provide but not limited to: clear multi-angle pictures of the suspension, packaging + batch number, time and place of the user, and a brief description of the use process.

- ***Can it be combined with optoelectronic projects/Botox?***

It can be combined, it is recommended to do PLLA injection first, and then do photoelectric projects after 4-6 months (photoelectric projects are generally 30-40 minutes, the temperature is high, and PLLA will increase the decomposition rate at 50-60 degrees Celsius, so if you do PLLA injection first Facial acupuncture, followed by photoelectric treatment, the effect of PLLA will be impaired; if you want to do it at the same time, you can stagger the parts, or perform photoelectric treatment first, and then perform Devolux treatment after partial skin cooling for half an hour.

It can also be combined with Botox treatment, because Botox can block the information conduction between nerves and muscles, which is more conducive to the retention of PLLA

